

# Topilase<sup>®</sup>

## Application protocol

- Press the upper compartment to release the enzymes into the solution
- Shake the bottle vigorously
- Once empty, discard the upper compartment and screw the dosing tip to the lower compartment
- Shake the bottle again
- Apply a few drops of Topilase<sup>®</sup> to the area to be handled and spread the product with your gloved hands
- Massage the area well for 3 to 5 minutes
- Remove excess product

## Care spacing

The number of sessions depends on:

- the desired result
- the type of hyaluronic acid injected
- the date on which the area was injected
- the plane in which the area was injected

Each Topilase<sup>®</sup> treatment should be spaced one week apart

On average 3 sessions will be necessary:

1st session on D-0 / 2nd session on D-7 / 3rd session on D-14

## Conservation

- Maximum 14 days in the refrigerator once opened

**Topilase**<sup>®</sup>  
is an innovative tool  
that allows you to  
adjust without risk the  
appearance of  
irregularities linked to  
hyaluronic acid  
injections. It is an  
essential tool, non-  
invasive, effective and  
without side effects.



Share your experience with  
Topilase<sup>®</sup> and tag



**@softfil\_international**



**SCAN ME**